## CONTENT

<b>EN (English)</b> Instruction Manual GenTek™ Hexalobular Screwdriver.....	2
<b>BG (Български)</b> Ръководство за употреба GenTek™ Шестостенна отвертка.....	4
<b>CS (Česky)</b> Návod k použití GenTek™ hexalobulární šroubovák.....	6
<b>DA (DANISH)</b> Brugervejledning GenTek™ Hexalobular Screwdriver.....	8
<b>DE (Deutsch)</b> Gebrauchsanweisung GenTek™ Sechsrund-Schraubendreher.....	10
<b>EL (Ελληνικά)</b> Εγχειρίδιο οδηγιών GenTek™ Εξάλοβο κατσαβίδι.....	12
<b>ES (Español)</b> Manual de instrucciones GenTek™ Destornillador Hexalobular.....	14
<b>ET (Eesti)</b> Kasutusjuhend GenTek™-i kuusnurkne kruvikeeraja.....	16
<b>FI (Suomi)</b> Käyttöopas GenTek™ kuusikulmainen ruuvimeisseli.....	18
<b>FR (Français)</b> Mode d'emploi Tournevis hexalobé GenTek™.....	20
<b>HR (Hrvatski)</b> Priručnik GenTek™ Heksalobularni Odvijač.....	22
<b>HU (Magyar)</b> Kezelési útmutató GenTek™ torx csavarhúzó.....	24
<b>IT (Italiano)</b> Istruzioni d'uso Cacciavite esalobato GenTek™.....	26
<b>LT (Lietuvių k)</b> Instrukcijų vadovas GenTek™ šešiaskiltis atsuktuvus.....	28
<b>LV (Latviski)</b> Lietošanas rokasgrāmata GenTek™ sešstūrveida skrūvgriezis.....	30
<b>NL (Nederlands)</b> Instructiehandboek Hexalobulaire GenTek™ schroevendraaier.....	32
<b>NO (Norsk)</b> Instruksjonshåndbok GenTek™ heksalobulær skrutrekker.....	34
<b>PL (Polski)</b> Instrukcja obsługi Wkręтак torx GenTek™.....	36
<b>PT (Português)</b> Manual de Instruções GenTek™ Chave de parafusos Hexalobular.....	38
<b>RO (Română)</b> Manual de instrucțiuni Şurubelniţă hexalobată GenTek™.....	40
<b>SK (Slovak/Slovakian)</b> Návod na použitie Skrutkovač na skrutky s hexalobulárnou drážkou GenTek™.....	42
<b>SL (Slovensčina)</b> Navodila za uporabo Heksalobularni izvijač GenTek™.....	44
<b>SV (Svenska)</b> Instruktionsmanual GenTek™ Hexalobular Skruvmejsel.....	46

# Instruction Manual

## GenTek™

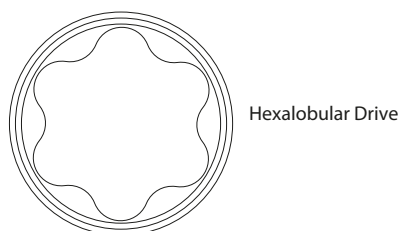
### Hexalobular Screwdriver

Version: 3 / 07. 2023  
 Compiled at: 07. 07. 2023  
 Effective date: 07. 07. 2023  
 Replaces version: 2 / 05. 2021

#### Indications for Use

The GenTek™ Hexalobular Screwdriver is used to tighten/loosen GenTek™ Hexalobular (laboratory try-in or final prosthetic) Screws to enable assembly/disassembly of GenTek™ single-unit (engaging) and multi-unit (non-engaging) restorative systems containing the hexalobular screw to corresponding Zimmer Biomet Dental Implant Systems.

Hexalobular Drive

#### Device description

The GenTek™ Hexalobular Screwdriver is used to drive the GenTek™ Hexalobular screw through a restorative screw access channel containing an angulation of up to 25°.

The screwdriver is designed with an ISO-latch connection allowing for use with Zimmer Biomet Low Torque Indicating Ratchet Wrench Standard ISO 1797 Adapter\* [Item #: C9980]. The Zimmer Biomet Low Torque Indicating Ratchet Wrench Standard ISO 1797 Adapter [Item #: C9980] is intended to be used with the Zimmer Biomet Low Torque Indicating Ratchet Wrench (L-TIRW).

The screwdriver is an individually packaged reusable component and is available in various lengths. The screwdriver is provided non-sterile and must be cleaned and sterilized prior to use in a patient's mouth.

#### Materials

GenTek™ Hexalobular Screwdriver are manufactured from medical grade stainless steel.

#### Procedure

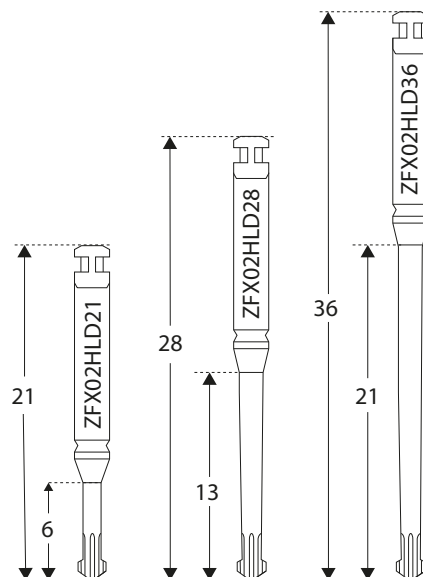
##### × Handling

1. Connect the ISO-latch portion of the driver into Low Torque Indicating Ratchet Wrench Standard ISO 1797 Adapter [Item #: C9980]
2. Engage the screwdriver to the corresponding GenTek™ Hexalobular Screw to tighten or loosen

Refer to the below table for placement torque recommendations for GenTek™ Hexalobular (laboratory try-in or final prosthetic) Screws:

Connection*	Angulation Range	Laboratory Try-in Screw	Final Prosthetic Screw Torque	GenTek™ Hexalobular Driver*
Certain®	0–25 degrees	Hand-tighten**	20 Ncm	ZFX02HLD21, ZFX02HLD28, ZFX02HLD36
TSV® / Trabecular Metal™	0–25 degrees	Hand-tighten**	30 Ncm	ZFX02HLD21, ZFX02HLD28, ZFX02HLD36
Eztetic®	0–25 degrees	Hand-tighten**	30 Ncm	ZFX02HLD21, ZFX02HLD28, ZFX02HLD36
External Hex	0–25 degrees	Hand-tighten**	32 Ncm	ZFX02HLD21, ZFX02HLD28, ZFX02HLD36

\*Applicable to single-unit and multi-unit for each connection  
 \*\*Minimum of 5 Ncm; Maximum of 15 Ncm

##### × Cleaning and Sterilization

The GenTek™ Hexalobular Screwdriver is non-sterile. Prior to use, the driver must first be cleaned and sterilized. The driver can be used up to 15 cleaning/sterilization cycles.

#### Cleaning

1. Rinse the driver under running cold tap water for 2 minutes.
2. Hand wash instruments in a bath with demineralized water for 5 minutes. Remove debris with a soft brush. Cavities should be rinsed with a syringe. This step should be repeated until all debris is removed.
3. Clean the instrument in an ultrasonic bath using a mildly alkaline cleaner (1.5%) at a frequency of at least 40 kHz for 20 minutes.
4. Rinse thoroughly with demineralized water for 1 min.

#### Sterilization

The recommended sterilization procedure is full cycle pre-vacuum steam sterilization at a temperature of 134 °C (273.2 °F) for 3 minutes, 4 pulse minimum or full cycle gravity steam sterilization at 134 °C (273.2 °F) for 3 minutes minimum. The products are ready for use 5 minutes after completion of the autoclaving process.

**Note:** Follow the loading instructions issued by the sterilizer manufacturer.

#### Storage and Handling

The GenTek™ Hexalobular Screwdriver should be stored at room temperature.

#### Contraindications

None

#### Known Side Effects

None

# Instruction Manual

## GenTek™

### Hexalobular Screwdriver

Version: 3 / 07. 2023  
Compiled at: 07. 07. 2023  
Effective date: 07. 07. 2023  
Replaces version: 2 / 05. 2021

#### Warnings

- × The GenTek™ Hexalobular Screwdriver must only be used and handled by dental professionals who have approved training in dentistry.
- × GenTek™ Hexalobular Screwdriver may only be used for GenTek™ Hexalobular Screws.
- × The GenTek™ Hexalobular Screwdrivers are small and, therefore, they must be handled with caution to avoid swallowing or inhaling by the patient.
- × The use of a screw torque higher than the recommended value may result in damage to the GenTek™ Hexalobular Screwdriver, GenTek™ TiBase, screws, custom abutment, and/or the implant.
- × The screwdriver must be in good condition. A worn screwdriver can lead to over-rotation of the hexalobe in the screw head which can result in the screw not being fully tightened and/or not being able to be removed.

#### Explanation of symbols

Non-sterile

Do not use if package is damaged

Manufacturer Information: Address of legal manufacturer facility

Unique Device Identifier

LOT-Number, Symbol followed by Production Date of the lot.

Medical Device

Reference Number, Symbol followed by Manufacturer's Item Number

Quantity/ Unit of package content

Consult instructions for use  
[www.zfx-dental.com](http://www.zfx-dental.com)

Consult instructions for use for mating hexalobular screws and angulated screw channel TiBase abutments

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Zfx GmbH