

perio precision



Periodontal maintenance



P2L

Debridement of the premolar and molar sectors, left-oriented

Round micro-tip recommended for periodontal debridement in the presence of a fine periodontium and in narrow areas.

- Maxillary: buccal and distal surfaces of sector 2, pivots at 13, then the palatine and mesial surfaces of sector 1.
- Mandibular: buccal and distal surfaces of sector 4, pivots at 43, then lingual and mesial surfaces of sector 3.



P2L
F00090



P2R

Debridement of the premolar and molar sectors, right-oriented

Second instrument in the sequence, after the P2L tip.


The double bend makes it possible to treat areas that are difficult to access (inter-radicular spaces, deep pockets).

- Maxillary: buccal and mesial surfaces of sector 2, pivots at 13, then buccal and distal surfaces of sector 1.
- Mandibular: lingual and mesial surfaces of sector 4, pivots at 43, then buccal and distal surfaces of sector 3.



P2R
F00091

The P2 tips can also be used to remove small amounts of excess cement when bonding fixed prosthesis.

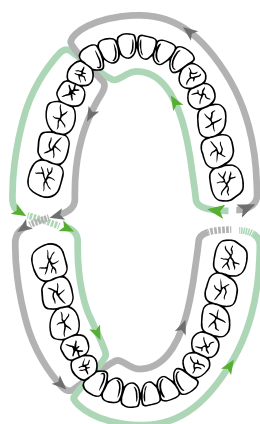

→ TK2-1L / P2L
→ TK2-1R / P2R