

PRESIDENT® Disposable Impression Trays


Indications for Use – To hold impression materials – use to make impressions of patients’ teeth.


Instructions for Use

The PRESIDENT® Disposable Impression Trays provide the rigidity of a metal tray with the convenience of a plastic tray. The Disposable Impression Tray is specifically designed to provide the highest degree of rigidity and stability to prevent distortion and bite-through to achieve consistently accurate impressions. It is made out of a highly rigid, transparent polystyrene plastic. The tray is anatomically designed to fit most patients and can be adjusted if necessary.

Procedure:

- Try tray in patient’s mouth to check fit and make adjustments if needed.
- Easily trimmed or flame softened for adjustment to customize fit.
 - To Soften: pass tray through a flame a few times to soften and then bend to desired shape.
 - To Trim: use an arbor band, large size bur, or any acrylic cutter to trim to the desired size. Be careful to leave smooth edges after trimming.
- For additional retention of silicone based impression materials, the use of tray adhesive is recommended.
 - Paint tray adhesive on the inside surface and peripheral area of the tray. Allow tray adhesive to dry according to manufacturer's instructions.
- Prior to initial use, tray can be disinfected by a glutaraldehyde base - cold solution method. Follow disinfectant manufacturer's instructions.

 CAUTION	Federal law restricts this device to sale by or on the order of a licensed a healthcare practitioner.
	Do not use if product has melted.
	Dispose of according to local regulations.

 MANDATORY SINGLE USE ONLY due to contact with the oral environment.

 www.coltene.com

Coltène/Whaledent Inc. 
 235 Ascot Parkway
 Cuyahoga Falls, OH 44223 / USA
 Tel. USA & Canada 1 800 221 3046
 +1 330 916 8800
 Fax +1 330 645 8704
 info.us@coltene.com

Coltène/Whaledent GmbH + Co. KG 
 Raiffeisenstrasse 30
 89129 Langenau / Germany
 Tel. +49 (0) 7345 805 0
 Fax +49 (0) 7345 805 201
 info.de@coltene.com

President® Impression Trays		
C6901 through C6909 contain 12 disposable perforated plastic trays		
C6901	Upper Arch - Large	Size 1
C6902	Lower Arch - Large	Size 2
C6903	Upper Arch - Medium	Size 3
C6904	Lower Arch - Medium	Size 4
C6905	Upper Arch - Small	Size 5
C6906	Lower Arch - Small	Size 6
C6907	Upper Left/Lower Right Quadrant	Size 7
C6908	Upper Right/Lower Left Quadrant	Size 8
C6909	Anterior Universal	Size 9
C6910	Assortment Pack (1 of each (9) Sizes)	All

P/N 20005064B

 Copyright ©2019
 Made in USA


PRESIDENT® Tray AC

Autoclavable Disposable Impression Trays

Indications for Use – To hold impression materials – use to make impressions of patients’ teeth.

Instructions for Use


The PRESIDENT® Disposable Impression Trays provide the rigidity of a metal tray with the convenience of a plastic tray. The Disposable Impression Tray is specifically designed to provide the highest degree of rigidity and stability to prevent distortion and bite-through to achieve consistently accurate impressions. It is made out of a highly rigid, blue polycarbonate plastic. The tray is anatomically designed to fit most patients and can be adjusted if necessary.


Procedure:

- Try tray in patient’s mouth to check fit and make adjustments if needed.
- Easily trimmed or flame softened for adjustment to customize fit.
 - To Soften: pass tray through a flame a few times to soften and then bend to desired shape.
 - To Trim: use an arbor band, large size bur, or any acrylic cutter to trim to the desired size. Be careful to leave smooth edges after trimming.
- For additional retention of silicone based impression materials, the use of tray adhesive is recommended.
 - Paint tray adhesive on the inside surface and peripheral area of the tray. Allow tray adhesive to dry according to manufacturer's instructions.
- Prior to initial use, autoclave the tray in accordance with the manufacturer’s instructions

Sterilization:

Autoclave: President Tray AC impression trays should be autoclaved with a gravity steam sterilization cycle. Autoclave in an FDA approved pouch at 132° C (270° F) for 15 minutes.

 CAUTION	Federal law restricts this device to sale by or on the order of a licensed a healthcare practitioner.
	Do not use if product has melted.
	Dispose of according to local regulations.

 MANDATORY SINGLE USE ONLY due to contact with the oral environment.

 www.coltene.com



Coltène/Whaledent Inc. 
 235 Ascot Parkway
 Cuyahoga Falls, OH 44223 / USA
 Tel. USA & Canada +1 800 221 3046
 +1 330 916 8800
 Fax +1 330 645 8704
 info.us@coltene.com

Coltène/Whaledent GmbH + Co. KG
 Raiffeisenstrasse 30
 89129 Langenau / Germany
 Tel. +49 (0) 7345 805 0
 Fax +49 (0) 7345 805 201
 info.de@coltene.com

President® Tray AC		
C6950 through C6959 contain 12 disposable perforated plastic trays		
C6950	Upper Arch - Large	Size 1
C6951	Lower Arch - Large	Size 2
C6952	Upper Arch - Medium	Size 3
C6953	Lower Arch - Medium	Size 4
C6954	Upper Arch - Small	Size 5
C6955	Lower Arch - Small	Size 6
C6956	Upper Left/Lower Right Quadrant	Size 7
C6957	Upper Right/Lower Left Quadrant	Size 8
C6958	Anterior Universal	Size 9
C6959	Assortment Pack (1 of each (9) Sizes)	All

P/N 20005064B