

Zeta 7 Spray



Special applications

A ready-to-use disinfectant spray for the rapid disinfection of impressions.

Active ingredients: alcohols.



PRODUCT BENEFITS

- ▶ High protection: bactericidal, yeasticidal, tubercolicidal and virucidal.
- ▶ Compatibility with materials for the impression taking (addition and condensation silicones, alginate, polyether, polysulphide and polyvinyl).
- ▶ Safe to use: free from aldehydes and phenols.
- ▶ Practical, fast and ready to use, without rinsing, disinfects up to 250 impressions.

Use

Rinse the impression under running water. Spray **Zeta 7 Spray** onto both sides of the impression to obtain a continuous film of disinfectant covering the two surfaces. Leave to dry for at least 3 minutes.

RESPECTS THE DIMENSIONAL STABILITY OF IMPRESSIONS.

Spectrum



Bactericidal: EN 13727 (S. aureus, P. aeruginosa, E. hirae)

Yeasticidal: EN 13624 (C. albicans)

Tubercolicidal: EN 14348, EN 14563 (M. terrae)

Virucidal: EN 14476 (tested on Poliovirus, Adenovirus, Norovirus, Parvovirus, including HIV, HBV, HCV)

Tests carried out in dirty conditions.