



Information guide

unica anterior



powered by:

STYLE ITALIANO™

matrix
systems




polydentia

swiss manufacture 



"In today's conservative dentistry, the choice of products and the techniques we apply everyday must be predictable and cost-effective, incorporating clear-cut advantages to our daily procedures. Unica is the solution to everyday anterior restorations, whether it be class III, IV, V, shape modifications, and/or composite veneers in direct stratification technique. One effective matrix for everyday anterior aesthetic and functional results."

Dr. Jordi Manauta, DDS
Head of StyleItaliano Idea Factory
Sestri Levante, Italy

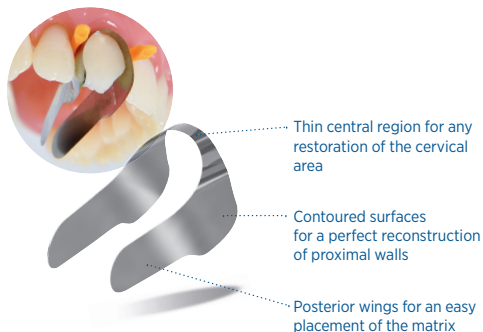

Index

| | |
|-----------------------|---|
| Class III Restoration | 3 |
| Class IV Restoration | 4 |
| Class V Restoration | 5 |
| Composite Veneers | 6 |
| Shape modification | 7 |
| Refill | 8 |

Class III restoration



By courtesy of
Dr. Dan Lazar
Romania



1. Pre-operative situation



2. The cavities after preparation
(cleaning, caries removal and
disinfection)



3. Positioning of Unica anterior



4. The cavity filled with
composite



5. Post-operative view of the
restoration



*Unica may be cut to fit
only one proximal surface*



for more
information

Class IV restoration



By courtesy of
Dr. Dan Lazar
Romania



for more
information



1. Pre-operative situation



2. Surfaces preparation and
cleaning



3. Positioning of Unica anterior



4. Tooth surface restoration



5. Post-operative view of the
restoration



*Unica may be cut to fit
only one proximal surface*

Class V restoration



By courtesy of
Dr. Giuseppe Chiodera
Italy



1. Pre-operative situation



2. The cavity after preparation
(cleaning, caries removal and
disinfection)



3. Positioning of Unica anterior



4. The cavity filled with
composite



5. Immediate post-operative
view of the restoration



for more
information

Composite veneers



By courtesy of
Dr. Monaldo Saracinelli &
Dr. Antonino Nicolò
Italy



1. Pre-operative situation



2. Surface preparation and
positioning of Unica anterior



3. Tooth surface restoration



4. Clinical situation before
finishing and polishing



5. Post-operative view of the
restoration



for more
information


Shape modification



By courtesy of
Dr. Carlos Fernández Villares
Madrid, Spain



1. Pre-operative situation

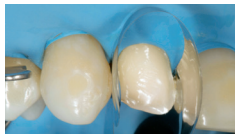

2. Tooth surface preparation



3. Positioning of Unica anterior



4. Tooth surface build up



5. Post-operative view of the
restoration



for more
information








Unica Introkit anterior

REF 6900

- 35 pcs Unica anterior matrices
- 40 pcs myWedge XS
- 40 pcs myWedge S
- 40 pcs myWedge M
- 1 pc myCustom Resin, 3 ml syringe
- 7 pcs needle tips gauge 18 for myCustom Resin
- 1 pc myQuickmatrix Forceps



Refill

| | Quantity | REF |
|--|----------|------|
|  Unica anterior matrices | 12 | 6912 |
|  Unica anterior matrices | 50 | 6950 |
|  myWedge, Extrasmall (XS) | 100 | 7101 |
|  myWedge, Small (S) | 100 | 7102 |
|  myWedge, Medium (M) | 100 | 7103 |
|  myCustom Resin, 3ml syringe & 10 pcs needle tips gauge 18 | | 6811 |
|  myQuickmatrix Forceps | | 5703 |

Swiss Made
Polydentia SA
via Cantonale 47
6805 Mezzovico-Vira
Switzerland

phone: +41 91 946 29 48
fax: +41 91 946 32 03

info@polydentia.ch
www.polydentia.ch



Visit "Unica Community"
to share
Unica anterior
clinical cases

Follow us
#thinkunica



REV 01.05.2018



polydentia
swiss manufacture 