

One single procedure for your crown cementations


Zirconia, Lithium Disilicate or Metal. No primers required.
One single procedure regardless of the restorative material.



Conditioning the crown.*
* Refer to "Conditioning the prosthetic restoration".



Only for Handmix
10 sec.
Do not store the mixing pad or spatula in the refrigerator to avoid condensation.
Dispense an equal amount of Paste A & B.
Mix Paste A & B for 10 seconds.

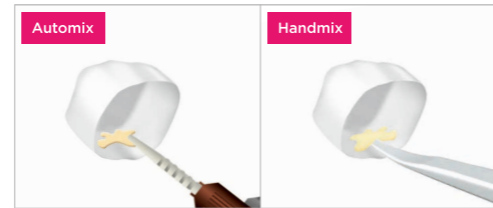

Apply the cement to the crown.



Place the crown.




Light-cure 2-5 sec.
Chemical-cure 2-4 min.
Light-cure for 2 to 5 seconds or chemical-cure for 2 to 4 minutes, then remove the excess cement.



5 min.
Maintain isolation for 5 minutes.*
* For a translucent restoration, light-cure.

Conditioning the prosthetic restoration

Follow the instructions for use of the restoration material.
In the absence of specific instructions, we recommend the following procedure:

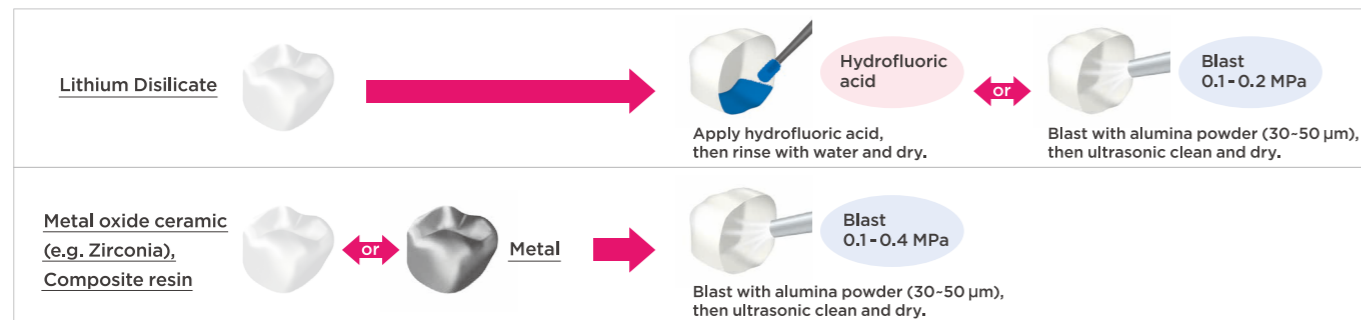

PANAVIA™ SA Cement Universal Automix

· Universal (A2)
· White
· Translucent
Content: 1 syringe:
Paste A and B: 8.2 g (4.6 mL), Mixing Tip: 20 tips



PANAVIA™ SA Cement Universal Handmix

· Universal (A2)
· White
· Translucent
Content: 1 syringe:
Paste A and B: 9.2 g (5 mL)

- Before using this product, be sure to read the Instructions for Use supplied with the product.
- The specifications and appearance of the product are subject to change without notice.
- Printed color can be slightly different from actual color.

"PANAVIA", "AVENCIA" and "CLEARFIL" are trademarks of Kuraray Co., Ltd.
"KATANA" is a trademark of NORITAKE CO., LIMITED.

PANAVIA™ SA Cement Universal

SIMPLE, CLEAN AND VERSATILE

FORGET ABOUT PRIMERS



Kuraray Noritake Dental Inc.

1621 Sakazu, Kurashiki, Okayama 710-0801, Japan

Website www.kuraraynoritake.com

US Distributed by
Kuraray America, Inc.
33 Maiden Lane, 6th Floor
New York, NY 10038

EC REP
Kuraray Europe, GmbH
Philipp-Reis-Str. 4
65795 Hattersheim am Main, Germany

R Only **0120**

OST-27
LW20-01 mm/2019

Kuraray Noritake Dental Inc.

www.kuraraynoritake.com