
3M

3M | Pentamix™ 3

ESPE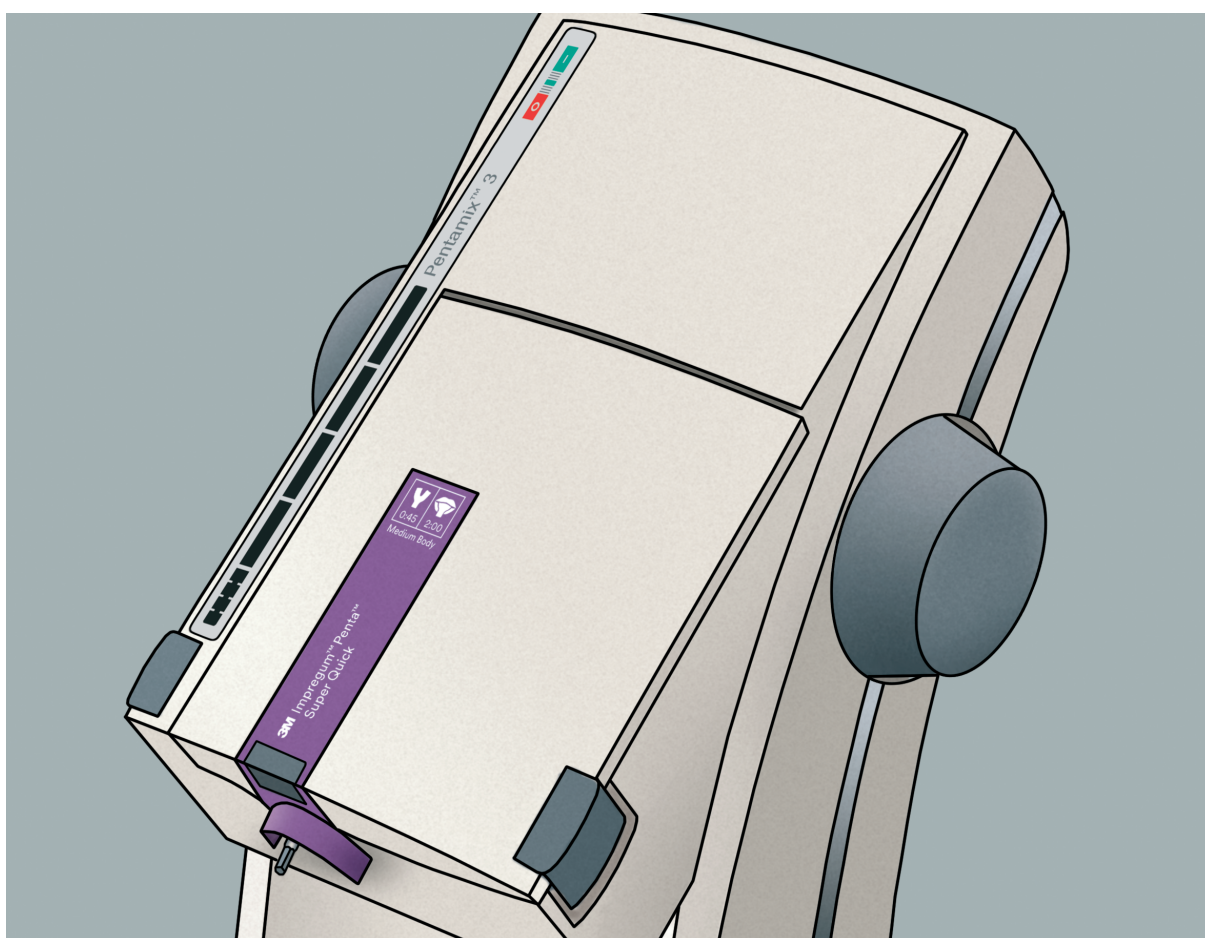

- (en) Automatic Mixing Unit
- (ru) Аппарат Pentamix 3 для автоматического смешивания стоматологических оттисковых материалов
- (bg) Автоматична бъркачка
- (hr) Uređaj za automatsko miješanje
- (hu) Automata keverő berendezés
- (pl) Urządzenie do automatycznego mieszania mas wyciskowych
- (ro) Aparat pentru malaxarea automată a materialelor de amprentă
- (sk) Automatický zmiešavač
- (sl) Avtomatski mešalec
- (cs) Automatický míchací přístroj
- (tr) Otomatik Karıştırma Cihazı
- (et) Jäljendmaterjalide segamiseade
- (lv) Automātiska maisīšanas iekārta
- (it) Automatiche maišyklė
- (uk) Апарат для автоматичного замішування

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Operating Instructions

Table of Contents

	Page
1. Safety Instructions	5
2. Product Description	5
2.1 Comparison with Pentamix 2	6
3. Technical Data	6
4. Installation	6
4.1 Tabletop Installation	6
5. Operation	6
5.1 Loading the Cartridge	6
5.2 Inserting the Cartridge	6
5.2.1 Inserting the Pentamix 3 Cartridge	6
5.2.2 Inserting and Removing a Pentamix 2 Cartridge	6
5.3 Attaching the Mixing Tip	7
5.3.1 Attaching the Mixing Tip to a Pentamix 3 Cartridge	7
5.3.2 Attaching the Mixing Tip to a Pentamix 2 Cartridge	7
5.4 Starting a New Cartridge	7
5.5 Loading the Impression Tray and Penta Elastomer Syringe	7
5.6 Re-starting a Partly Empty Cartridge	7
5.7 Replacing a Cartridge	7
5.8 Storage of Partially Used Cartridges	7
5.9 Fill Level Gauge	7
5.10 Disposal of Consumable Materials	7
5.11 Notes	7
6. Malfunctions and Error Messages	8
7. Maintenance and Care	8
7.1 Replacement of the Plunger Discs	8
7.2 Care	8
7.3 Maintenance and Repair	8
8. Return of Old Electric and Electronic Equipment for Disposal	8
8.1 Collection	8
8.2 Return and Collection Systems	8
9. Customer Information	9
9.1 Warranty	9
9.2 Limitation of Liability	9
Symbol Glossary	9

1. Safety Instructions

ATTENTION! Read these Operating Instructions carefully before plugging in and operating the unit! As with all technical units, the trouble-free function and safe operation of this unit can be assured only if standard safety procedures and the special safety instructions in these Operating Instructions are observed.

3M Deutschland GmbH guarantees trouble-free operation of the Pentamix system only if 3M system components are used exclusively.

- The unit may be used only in strict accordance with the following instructions. We accept no liability for any damage arising from any other or improper use of this unit.
- If the unit is moved from a cold into a warm environment, condensation can result in a hazardous situation. Never begin operating the unit until it has reached the ambient temperature.
- The power plug must be connected into a suitably grounded socket before the unit is operated. If an extension cable is used, make sure that it also provides a ground wire.
- Position the unit so that the power plug is accessible at all times.
- The unit must be placed either on a non-slip, level surface or mounted to a sufficiently stable wall to insure stable placement.
- Use only Penta™ materials in foil bags in combination with the Penta™ cartridges and Penta™ mixing tips.
- Close the cover of the unit before operating.
- Do not lock the "Start" button as this can lead to hazardous situations and undesired continuous operation.
- Always turn off the unit and disconnect the plug from the power source before changing the plunger discs.
- When changing the plunger discs, position the smooth side of the new discs towards the foil bag (the opposite side of the discs with the fan-shaped depressions must face away). Otherwise, the foil bag might be damaged.
- To avoid electrical shock, do not insert any objects into the unit.
- Use only genuine 3M parts when replacing defective components as directed in these Operating Instructions. We assume no liability for any damage caused by the use of non-3M parts.
- If, for any reason, it is possible that the safety of a unit has been compromised, that unit must be removed from operation and identified in such a way that it is not inadvertently operated again by a third party. Safety may have been compromised, for example, if the unit does not work as specified or is visibly damaged.
- Keep solvents, flammable liquids, and powerful heat sources away from the unit as they may damage the plastic housing.
- Do not allow any cleaning agents to enter the unit during cleaning as they could cause an electrical short or a dangerous malfunction.
- Only a properly trained, authorized technician may open the unit housing and repair the unit.
- FDA studies have indicated that medical devices using radio frequency identification (RFID) may possibly affect the function of heart pacemakers and ICDs. However, as of this time the FDA has not received any reports of such effects. Moreover, the limited transmission range and the transmission frequency used by the Pentamix 3 unit mean that risks to users and patients are extremely improbable.

Please report a serious incident occurring in relation to the device to 3M and the local competent authority (EU) or local regulatory authority.

2. Product Description

Pentamix™ 3 is an automatic mixing unit for the preparation of Penta™ impression materials. 3M Penta impression materials have been specifically designed for this unit and are supplied in special foil bags. This is the only form of packaging which can be used in the Pentamix units. Every Penta™ foil bag is sealed with a **PentaMatic** sealing cap. The external geometry of the sealing cap insures its proper positioning on the cartridge. When the Pentamix unit is turned on, a special internal device opens the foil bag automatically once sufficient pressure has been built up by the plunger. The connection pieces located on the **PentaMatic** sealing caps serve for proper attachment of the mixing tip.

The cartridges and the **PentaMatic** sealing caps act together as a stable unit for the preparation of pastes in the Pentamix unit and allow a change of products without intermediate cleaning of the device and no loss of paste.

The Pentamix 3 unit is compatible with all Penta cartridges and Penta mixing tips. Steel-reinforced Penta cartridges (recognizable by the metal ring) and red Penta mixing tips are required to exploit the full extent of the performance capability of the Pentamix 3 unit (e.g., maximum mixing speed). When cartridges made completely of plastic and materials which do not satisfy requirements are used, the unit automatically adjusts the mixing speed as a precaution.

The Pentamix 3 is supplied as a tabletop unit, but can also be mounted on a wall using the commercially available brackets.

The product may only be used when the product labeling is clearly readable.

These Instructions for Use should be kept for the duration of product use.

For details on all additionally mentioned products please refer to the corresponding Instructions for Use.

Intended Purpose

Intended users: educated dental professionals, i.e. general dentists, dental assistants, dental hygienists, who have theoretical and practical knowledge on usage of dental products.

2.1 Comparison with Pentamix 2

- The Pentamix 3 unit can be disconnected from the power supply by using the main switch on the upper side of the housing (Fig. 7).
- A green LED light on the upper side of the housing shows that the unit is ready for operation (Fig. 7). The LED light blinks during the self-test when the unit is turned on and whenever the operating condition does not allow safe clinical use.
- The Pentamix 3 unit can mix pastes twice as fast as the Pentamix 2. Pentamix 3 automatically recognizes if the appropriate Penta foil bags with safety label or metal-reinforced Penta cartridges have been inserted. Pentamix 3 sets the optimal mixing parameters for the specific material being used.
- When the "Start" button is released, the Pentamix 3 unit automatically relieves the tension on the foil bags by withdrawing the plungers, which is indicated audibly by the sound of the drive motor.
- Safe operation at higher mixing speeds cannot be guaranteed unless Penta foil bags with safety labels and metal-reinforced Penta cartridges are used. This is why the Pentamix 3 unit has been designed to adjust mixing parameters to correspond to the mixing speed of the Pentamix 2 unit when other components are used.
- The locking lever of the inserted Penta 3 cartridge can be opened when the unit cover is closed so that the mixing tip can be replaced even when the cover is closed (Fig. 14).
- Since the locking lever of the Penta 3 cartridge snaps into a fully open position, only one hand is required to change the mixing tip (Fig. 16).
- Hand wheels and "Start" buttons have been placed on both sides of the Pentamix 3 unit so that it can be used easily by both right-handed and left-handed persons.
- The insertion of the cartridge into the Pentamix 3 unit is simplified by the plungers snapping into place in the uppermost position.

3. Technical Data

Operating mode:	Short-term operation – max. 1.5 min. on, min. 10 min. off
Cartridges:	Suitable for use with Penta foil bags Dimensions: 60 x 130 mm, 26.8 x 130 mm
Housing:	All parts of the housing are made of impact-resistant plastic (polycarbonate)
Power supply:	100 V – 240 V
Frequency:	47 Hz – 63 Hz
Power input:	max. 2 A
Ambient temperatures:	18° C/64° F..... 40° C/104° F
Rel. humidity:	20% 80%
Classification:	Hazard Class I
Dimensions:	L 283 mm x W 225 mm x H 280 mm
Weight:	9.8 kg
Storage and transport:	–20° C to +60° C/–4° F to +140° F Maximum rel. humidity 80%
Continuous sound pressure level:	>70 dB(A)
Year of manufacture:	See rating plate

4. Installation

The Pentamix 3 unit can be placed on a non-slip, level surface or mounted on a suitably strong wall. A special wall bracket is commercially available for mounting (Ident. No. 77601).

4.1 Tabletop Installation

- ▶ To connect the power cable, use the hand wheel to retract the mixing shaft.
- ▶ Close the cover and lay the unit on a suitable pad with the front of the unit facing down (Fig. 1).
- ▶ Remove the cover on the back of the housing by pressing the two snap locks (Fig. 2) in the direction of the arrows (Fig. 3).
- ▶ Thread the power cable through the opening in the cover and plug it into the power socket on the unit (Fig. 4).
- ▶ Replace the cover on the back of the unit. Make sure that the noses of the cover slide firmly into the openings in the housing (Fig. 5) and that the snap locks audibly click into place (Fig. 6).

Caution: Never operate the unit without replacing the cover; otherwise, the stability of the unit cannot be assured!

- ▶ Insert the power cable toward the back in the cable feedthrough.
- ▶ Insert the power cable of the Pentamix 3 unit into a grounded electrical socket. If an extension cable is used, make sure that it also provides a ground wire.
- ▶ Place the Pentamix 3 unit on a non-slip, level surface and press the main switch on the upper side of the unit to turn it on (Fig. 7). After blinking several times, the operating light burns steadily to indicate the unit is ready for operation.

5. Operation

5.1 Loading the Cartridge

Please carry out the steps in the following sequence exactly.

- ▶ Have ready one foil bag of base paste and one of catalyst paste as well as a matching, steel-reinforced 3M cartridge for a 3M Penta impression material. The colors on the foil bag cap and the locking lever or the product names on the foil bag and the locking lever must be the same.
- ▶ Insert the foil bags into the unlocked cartridge. The locating lugs make sure they are correctly seated (Fig. 8).
- ▶ Press down on the cartridge locking lever until it audibly clicks into place (Fig. 9).

5.2 Inserting the Cartridge

The Pentamix 3 unit works with both Pentamix 3 cartridges (recognizable by the locking lever) and the older Pentamix 2 cartridges (recognizable by the handle on the side).

5.2.1 Inserting the Pentamix 3 Cartridge

- ▶ Open the unit cover (Fig. 10).
- ▶ Use the hand wheel to retract the plungers until they stop. The plungers now snap into the uppermost position, and it is not necessary to hold them in place with the hand wheel.
- ▶ Hold the cartridge by the handle and insert it into the unit parallel to the unit opening (Fig. 11).
- ▶ Close the unit cover.
- ▶ Use the hand wheel to move the plungers forward until the plungers meet marked resistance from the foil bags (Fig. 12). The mixing process cannot be started until the plungers are touching the foil bags.
- ▶ Attach a new mixing tip prior to beginning mixing (see Section 5.3).

5.2.2 Inserting and Removing a Pentamix 2 Cartridge

- ▶ Open the unit cover.
- ▶ Use the hand wheel to retract the plungers until they stop. The plungers now snap into the uppermost position.
- ▶ Insert the cartridge into the unit parallel to the unit opening.
- ▶ Use the hand wheel to move the plungers forward until the plungers meet marked resistance from the foil bags. The mixing process cannot be started until the plungers are touching the foil bags.

- ▶ Attach a mixing tip prior to begin mixing (see Section 5.3).
- ▶ To remove the Pentamix 2 cartridge from the unit, use one finger to lift it by the cartridge handle from below.

5.3 Attaching the Mixing Tip. Always Attach a Mixing Tip Before Use!

Foil bags equipped with a **PentaMatic** sealing cap may be used exclusively with 3M Penta mixing tips.

5.3.1 Attaching the Mixing Tip to a Pentamix 3 Cartridge

- ▶ Release the locking lever by pushing up the lock (Fig. 13) and push the lever all the way up until it snaps into place (Fig. 14).
- ▶ First place the mixing tip with the CE symbol pointing upwards only on the tip of the mixing shaft (Fig. 15). Turn slightly to align the mixing tip with the corresponding outflow openings of the foil bags (Fig. 16). Attach the mixing tip and press down firmly to insure that the mixing tip is seated tightly on the outflow openings (Fig. 17).
 - Caution: Never attach the mixing tip only to the outflow openings of the foil bags when the mixing shaft has been withdrawn. This could prevent the full insertion of the mixing shaft into the mixing tip. In this case, the operating light will blink and the unit will not start (see Section 6).
- ▶ Press the locking lever down until it audibly snaps into place (Fig. 18).

5.3.2 Attaching the Mixing Tip to a Pentamix 2 Cartridge

- ▶ Open the unit cover.
- ▶ Release the lock.
- ▶ First place the mixing tip with the CE symbol pointing upwards only on the tip of the mixing shaft (Fig. 15). Turn slightly to align the mixing tip with the corresponding outflow openings of the foil bags (Fig. 16). Attach the mixing tip and press down firmly to insure that the mixing tip is seated tightly on the outflow openings (Fig. 17).
- ▶ Press the locking lever down until it audibly clicks into place (Fig. 18).
- ▶ Close the unit cover.

5.4 Starting a New Cartridge

- ▶ Press the "Start" button (Fig. 19). The Pentamix 3 unit can be operated using either the left-hand or right-hand "Start" button.
- ▶ When a new pair of foil bags is started, please wait **5-15 sec.** until pressure has built up in the cartridge; the foil bags will then open automatically.
- ▶ Once the bags are open, one of the two pastes, which are of different colors, may flow out first, producing an uneven mixture. For this reason, **observe the flow of the two pastes, discard any uneven mixture, and do not apply the paste until the mixture shows a homogeneous color** (Fig. 19).
- ▶ After mixing, leave the mixing tip on the cartridge as a seal!

5.5 Loading the Impression Tray and Penta Elastomer Syringe

- ▶ Press the "Start" button. The Pentamix 3 unit can be operated using either the left-hand or right-hand "Start" button.
 - The unit extrudes paste only as long as the "Start" button is held down. When the "Start" button is released, the motor audibly continues to run briefly. This movement relieves the pressure on the foil bags to minimize the unnecessary flow of the pastes.
 - Do not lock the "Start" button as this can lead to hazardous situations and undesired continuous operation. For safety reasons, use the unit only when the cover is closed.
- ▶ Tilt the tray to load it. Fill the tray continually from one side to the other, keeping the opening of the mixing tip immersed in the paste (Fig. 20 or 21). If necessary, use a spatula to smooth the impression material surface.
- ▶ When filling a Triple Tray®, fill one side first. Release the "Start" button briefly and turn the tray over, then fill the other side.
- ▶ A mixing tip may be used only as long as the mixing shaft inside the mixer rotates smoothly. If the mixing process is interrupted for more than 30 seconds, the material inside the mixing tip will begin to set, and a new mixing tip must be attached (see Point 5.3, "Attaching the Mixing Tip"). If the mixing unit is operated when there is set material inside it, the unit will turn off automatically to prevent damage.

- ▶ Attach the rear opening of the Penta Elastomer Syringe (after removing the plunger) to the mixing tip opening and press the "Start" button (Fig. 23).
 - CAUTION: Not all Penta impression materials can be syringed. Please refer to the Instructions for Use of the specific product.
- ▶ **Cleaning the Penta Elastomer Syringe**
Please refer to the pertinent Instructions for Use for information about cleaning and disinfecting the Penta Elastomer Syringe.

5.6 Re-starting a Partly Empty Cartridge

- ▶ After prolonged storage (especially for silicones), use an instrument to remove any plugs that may have formed from the outflow openings of the foil bags.
- ▶ It may be necessary to start the unit without a mixer to check the paste flow.
- ▶ Attach a new mixing tip and start the mixing procedure.

5.7 Replacing a Cartridge

If a cartridge is nearly empty, we recommend to have a second cartridge ready for use at hand so that it is possible to replace the exhausted cartridge quickly. This cartridge should be prepared as described under 5.4 "Starting a New Cartridge".

Experienced personnel can even replace a cartridge while filling a tray. You can save time by attaching the full mixing tip from the empty cartridge to the prepared new cartridge. Do not forget to move the plungers into contact with the bags!

Lifetime of the mixing tip: see 5.5.

5.8 Storage of Partially Used Cartridges

- ▶ Foil bags which have been started and partially emptied must be stored only in the cartridge and with attached and filled mixing tip.
- ▶ Store any cartridges with partially used foil bags either horizontally or with the mixing tip point down (Fig. 24). If stored improperly, the impression material may flow backwards, which would prolong the restarting time or cause the base and catalyst pastes to mix.

5.9 Fill Level Gauge

The fill level gauge shows how much material remains in the foil bags.

If the plungers have snapped into the uppermost position, the red fill level marker is not visible.

When the red marker reaches the first line, there is still enough impression material in the unit to fill completely either a large impression tray or three Triple Trays (Fig. 25).

When the foil bags are completely empty, the red fill gauge marker disappears, and the unit shuts off automatically.

5.10 Disposal of Consumable Materials

Dispose of used mixing tips and empty foil bags in residual waste. Be sure to comply with local waste disposal regulations.

The used-off-packaging of the Penta base paste is subject to the EU Waste Electrical and Electronic Equipment Directive (WEEE) and must be collected and disposed in the EU separately from other waste in accordance with the local regulations for electric or electronic waste.

5.11 Notes

Before using the foil bag, record the batch data printed on the bag (batch code, use-by date, 14-digit GTIN) for the office's own documentation or for questions to the manufacturer and archive the data so that they can later be accessed at any time.

Penta Mixing Tips - Red are single-use devices and would be damaged when removing set impression material.

6. Malfunctions and Error Messages

Malfunction	Cause	Remedy
The unit does not start	Foil bags empty	Insert new foil bags in the cartridge
The unit does not start and the operating light blinks	Plungers are not moved into the cartridge	Use the hand wheel to move the plungers up against the foil bags
	Mixing shaft does not fit tightly into the mixing tip	Remove the mixing tip and re-attach as described in 5.3
Unit starts, but no paste is extruded	New pair of foil bags	When a new pair of foil bags has been inserted, it takes about 5–15 sec. until the foil bags open from the pressure
	Plungers are not in contact with the foil bags	Use the hand wheel to move the plungers forward until the plungers meet marked resistance from the foil bags
	Paste temperature too low	Allow the pastes to warm to room temperature (minimum 18° C)
Paste is not extruded with the usual speed of the Pentamix 3	The inserted cartridge is not a metal-reinforced Penta cartridge	Use metal-reinforced Penta cartridge
	There is no safety label on the Penta foil bags, or it has been damaged	Check that the safety label is on the bags and is not damaged. Otherwise, please contact our Service Center
The cartridge cannot be inserted into the unit	Plungers are not in the uppermost position	Use the hand wheel to move the plungers up until they stop and hold
	The cartridge receptacle is dirty	Clean the cartridge receptacle, removing any foreign objects
Plungers cannot be moved into the cartridge	Cartridge not inserted correctly	Check the correct positioning of the cartridge in the Pentamix 3 unit
	Plunger disc defective	Check plunger discs for damage and replace if necessary (see 7.1)
The unit does not switch off when the "Start" button is released	The "Start" button is jammed	Pull the "Start" button up or, if necessary, use the main switch to turn off the unit and free the "Start" button
Hand wheels are difficult to turn back	Foil of a bag is wedged between the cartridge and the plunger	Run the plunger to the top position against the resistance. Check the plunger discs and replace if necessary (see 7.1)
Hand wheels completely blocked	Pressure on foil bags has not been released	Press the "Start" button briefly (1 sec.)
Operating LED does not light up	No power to the unit	Check the connection of the power plug at the unit and the outlet
	Main switch has not been turned on	Turn on main switch

7. Maintenance and Care

7.1 Replacement of the Plunger Discs

If damaged, the plunger discs must be replaced. Only original parts may be used as replacements to insure safe operation.

- ▶ Turn off the unit and disconnect the plug from the power source. Use an Allen wrench to remove the screw in the center of the plunger disc. Remove the disc and screw a new one into place (Fig. 26).
- ▶ When changing the plunger discs, make sure you position them with the correct alignment (see Fig. 26) to prevent damage to the foil bags.

7.2 Care

Clean all components with a soft cloth and, if necessary, a mild cleaning agent (e. g. dish-washing detergent). Solvents or abrasive cleaners may not be used in any case, as they can damage the plastic components!

- Cleaning agents must not enter the unit!

To disinfect all components, spray the disinfectant on a towel and use it to disinfect the unit. Do not spray the disinfectant directly on the device.

- Disinfectant agents must not enter the unit!
- Dry residual disinfectants on the device with a soft and fluff-free cloth, as they damage the plastic components.

If necessary, ask the manufacturer of the disinfectant if its constant use will damage plastic surfaces.

7.3 Maintenance and Repair

No components inside the unit require servicing or repair by the user. 3M Deutschland GmbH will accept responsibility for the safety, reliability, and correct function of the unit only if:

- Any modifications or repairs are carried out only by personnel authorized to do so by 3M Deutschland GmbH;
- The electrical fittings in the room comply with all regulatory requirements;
- The unit is operated according to these Operating Instructions.

Plastic cartridges are subject to wear depending on the material and frequency of use and are therefore not covered by the warranty for this unit. They should be regularly checked for signs of wear (e. g., fissures) and replaced with new, metal-reinforced Penta cartridges two years after initial operation at the very latest. The use of worn or defective cartridges can have adverse effects on the evenness of the mixture, reducing the quality of the impression.

8. Return of Old Electric and Electronic Equipment for Disposal

8.1 Collection

Users of electric and electronic equipment are required to collect their old equipment separately from other waste in accordance with the regulations of the specific country. Old electric and electronic equipment must not be disposed of with unsorted household waste. This separate collection is a prerequisite for recycling and reprocessing as an important method for preserving environmental resources.

8.2 Return and Collection Systems

When your Pentamix 3 is no longer usable, do not dispose of the device with household waste. 3M Deutschland GmbH has set up special disposal facilities to handle the equipment. Details about the procedure for the specific country can be obtained from the pertinent 3M Deutschland GmbH subsidiary.

9. Customer Information

No person is authorized to provide any information that deviates from the information provided in this instruction sheet.




9.1 Warranty

3M Deutschland GmbH warrants this product to be free from defects in material and manufacture. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES, INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH's sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

9.2 Limitation of Liability

Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

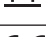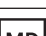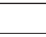Symbol Glossary

Reference Number and Symbol Title	Symbol	Description of Symbol
ISO 15223-1 5.1.1 Manufacturer		Indicates the medical device manufacturer as defined in EU Directives 90/385/EEC, 93/42/EEC and 98/79/EC.
ISO 15223-1 5.1.3 Date of Manufacture		Indicates the date when the medical device was manufactured.
ISO 15223-1 5.1.5 Batch code		Indicates the manufacturer's batch code so that the batch or lot can be identified.
ISO 15223-1 5.1.6 Catalogue number		Indicates the manufacturer's catalogue number so that the medical device can be identified.
ISO 15223-1 5.1.7 Serial number		Indicates the manufacturer's serial number so that a specific medical device can be identified.
ISO 15223-1 5.3.1 Fragile, handle with care		Indicates a medical device that can be broken or damaged if not handled carefully.
ISO 15223-1 5.3.4 Keep dry		Indicates a medical device that needs to be protected from moisture.
ISO 15223-1 5.4.4 Caution		Indicates the need for the user to consult the instructions for use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself.
ISO 7000-0623 This way up		Indicates to orientation to store and handle product.
CE Mark		Indicates conformity to European Union Medical Device Regulation or Directive.
Medical Device		Indicates the item is a medical device.
Rx Only		Indicates that U.S. Federal Law restricts this device to sale by or on the order of a dental professional. 21 Code of Federal Regulations (CFR) sec. 801.109(b)(1).
PAP 20		Indicates paper material is recyclable.
Low-density polyethylene		Indicates plastic component made of low-density polyethylene is recyclable.
Recycle electronic equipment		DO NOT throw this unit into a municipal trash bin when this unit has reached the end of its lifetime. Please recycle.
Green Dot		Indicates a financial contribution to national packaging recovery company per European Directive No. 94/62 and corresponding national law. Packaging Recovery Organization Europe.

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3M, ESPE, Penta and Pentamix are trademarks of 3M or 3M Deutschland GmbH.
Triple Tray is a registered trademark of Premier Products Co.
© 2019, 3M. All rights reserved.
3M, ESPE, Penta and Pentamix are trademarks of 3M or 3M Deutschland GmbH. © 2019, 3M. All rights reserved.