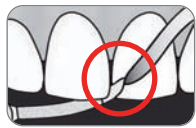
Proximal Finishing and Polishing Strip

**DO NOT
TWIST**

Twisting will cause strip to break. Use wedge if too tight.

1. Initial view of the clinical case.

2. Perforated strip ends for a secure and ergonomic grip.

3. The uncoated zone allows the introduction of the strip in tight proximal areas.

4. Beveling with the red working part, 40 µm

5. Beveling with the yellow working part, 15 µm

6. Finishing the composite filling with the red working part, 40 µm

7. Polishing the composite filling with the yellow working part, 15 µm

8. Final treatment/result

ENGLISH
INSTRUCTIONS FOR USE

For professional use only

GripStrip™**Product description**

- Metal strip, diamond-coated on one side, with two working parts of different grit sizes: 40 µm (red) and 15 µm (yellow)
- Perforated anatomical ends for better gripping
- Height: 2.5mm
- Thickness: 0.05mm
- Total length: 80mm
- Sterilizable and reusable

Indications for use

- Finishing and polishing of fillings and crown borders in interproximal areas.
- Removal of excess material and overhangs in fillings and crown borders.
- Beveling in adhesive restorations in cervical interproximal areas.

Practical Tips

- The total length of the strip (80mm) allows an optimal finger support on the adjacent teeth during work.
- To get past the contact area, use the uncoated part of the strip and exert a slight horizontal movement.
- After introduction, work with the less abrasive side (yellow, 15 µm) in order to facilitate the access of the more abrasive side (40 µm, red). Then begin the finishing procedure using a back-and-forth movement.
- Next, switch again to the less abrasive side (15 µm, yellow) to complete the polishing procedure.
- GripStrip must be removed with a vertical movement after repositioning it to the uncoated segment.
- In the event the contact area is very tight, it is useful to start by inserting a wooden wedge to open up a slight separation between the teeth before placing GripStrip in the area to be treated.

Maintenance and sterilization

- GripStrip is supplied non-sterile. Prior to first use on the patient and immediately after each use, the instruments must be disinfected, cleaned, and sterilized.
- Keep GripStrip separate from non-stainless steel instruments, such as rubber polishers and abrasive stones.
- If disinfection by immersion is being used, it is important to use only cleaning/disinfection solutions that provide corrosion protection and strictly observe the concentrations recommended by the manufacturer.
- Sterilization must be performed with validated procedures. If possible, use a single-pulsed or fractionated vacuum autoclave and subvacuum drying. Chemiclave sterilizers may also be used.

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Phone: +61 (0) 2.9006.1662© 2011 Centrix, Inc.