HySolate

Latex Dental Dam

Instructions for Use

Rx ONLY

 **COLTENE**

Table of Contents

EN	Instructions for Use	1
DE	Gebrauchsanweisung.....	10
FR	Mode d'emploi.....	19
ES	Instrucciones de uso	28
BG	Инструкции за употреба	37
CS	Návod k použití.....	46
DA	Brugsanvisning.....	55
EL	Οδηγίες χρήσης	63
ET	Kasutusjuhend	72
HR	Upute za upotrebu	81
HU	Használati útmutató	89
IT	Istruzioni per l'uso.....	98
LT	Naudojimo instrukcijos.....	106
LV	Lietošanas instrukcija	114
MT	Istruzzjonijiet għall-Użu	123
NL	Gebruiksaanwijzing.....	131
PL	Instrukcja użycia.....	140
PT	Instruções de utilização.....	149
RO	Instrucţiuni de utilizare	158
SK	Návod na použitie.....	166
SL	Navodila za uporabo	175
SV	Bruksanvisning.....	183
TR	Kullanma Talimatları.....	191

Intended Use/ Indications for Use:

Dental Dam is intended to be used intraorally to isolate one or more teeth and create a barrier to the surrounding area during a dental procedure.

Contraindications:

Latex Dental Dam should not be used on patients with a latex allergy.

Intended Users:

- Dental Practitioners
- Dental Assistants

Patient Target Group(s):

Suitable for all patient groups.

Clinical Benefits:

The use of Dental Dam has the following clinical benefits:

- Increased visibility through tooth/teeth isolation
- Reduced risk of the patient aspirating or swallowing instruments, tooth debris, medicaments or irrigation solutions
- Creation of a dry operating field
- Protection of soft tissue from contact with instruments
- Infection control barrier

Characteristics:

- Available in different colors and thicknesses
- Two sizes: 5" x 5" or 6" x 6" (127 mm x 127 mm or 152 mm x 152 mm)
- The lighter colors have the advantage of naturally illuminating the operating field whereas the darker colors can help with visual contrast

Storage:

Store Dental Dam between 32° F – 80° F (0°C – 26°C).

Keep dry and keep away from heat.

Residual Risks, Warnings and Precautions:

Refer to the Warnings and Precautions section for disclosure of risk associated with the use of the Dental Dam in line with its Intended Use.

Instructions

CAUTION! This Product Contains Natural Rubber Latex Which May Cause Allergic Reactions.

Dental Dam is intended to be used with various other devices including clamps and frames. Refer to the instructions for use for the other devices being used.

The following general steps are provided as guidance and can be tailored to fit the specific case:

Step 1: Determine the Operating Field.

Step 2: Dental Dam Preparation.

- Determine hole positions
- Mark hole positions
- Punch holes

Step 3: Clamp/Retainer Selection

Step 4: Clamp/Retainer Application

Step 5: Dental Dam Placement

Parts List

Note: Not all Catalog Numbers are available in all regions

Catalog #	Description
60034424	HySolate Latex Dental Dam Light thin 127 x 127, 52 pcs
60034425	HySolate Latex Dental Dam Light medium 127 x 127, 52 pcs
60034426	HySolate Latex Dental Dam Light heavy 127 x 127, 52 pcs
60034427	HySolate Latex Dental Dam Light thin 152 x 152, 36 pcs
60034428	HySolate Latex Dental Dam Light medium 152 x 152, 36 pcs
60034429	HySolate Latex Dental Dam Light heavy 152 x 152, 36 pcs
60034430	HySolate Latex Dental Dam Light x-heavy 152 x 152, 36 pcs
60034414	HySolate Latex Dental Dam Green thin 127 x 127, 52 pcs
60034415	HySolate Latex Dental Dam Green medium 127 x 127, 52 pcs
60034416	HySolate Latex Dental Dam Green heavy 127 x 127, 52 pcs
60034417	HySolate Latex Dental Dam Green x-heavy 127 x 127, 52 pcs
60034418	HySolate Latex Dental Dam Green thin 152 x 152, 36 pcs
60034419	HySolate Latex Dental Dam Green medium 152 x 152, 36 pcs
60034420	HySolate Latex Dental Dam Green heavy 152 x 152, 36 pcs
60034421	HySolate Latex Dental Dam Green x-heavy 152 x 152, 36 pcs
60034441	HySolate Latex Dental Dam Blue medium Child 127 x 127, 52 pcs
60034442	HySolate Latex Dental Dam Blue medium 152 x 152, 36 pcs
60034432	HySolate Latex Dental Dam Light medium 127 x 127, 364 pcs
60034422	HySolate Latex Dental Dam Green medium 127 x 127, 364 pcs

Catalog #	Description
60034431	HySolate Latex Dental Dam Light medium 152 x 152, 360 pcs
60034423	HySolate Latex Dental Dam Green medium 152 x 152, 360 pcs
60034433	HySolate Fiesta Latex Dental Dam assorted thin 152 x 152, 36 pcs
60034434	HySolate Fiesta Latex Dental Dam assorted thin 127 x 127, 52 pcs
60034435	HySolate Fiesta Latex Dental Dam assorted medium 127 x 127, 52 pcs
60034436	HySolate Fiesta Latex Dental Dam assorted medium 152 x 152, 36 pcs
60034437	HySolate Fiesta Latex Dental Dam assorted heavy 127 x 127, 52 pcs
60034438	HySolate Fiesta Latex Dental Dam assorted heavy 152 x 152, 36 pcs
60034443	HySolate Latex Dental Dam Blue heavy 152 x 152, 36 pcs
60034439	HySolate Latex Dental Dam assorted medium 127 x 127, 364 pcs
60034440	HySolate Latex Dental Dam assorted medium 152 x 152, 360 pcs
60031355	HySolate Black Printed medium 152 x 152, 360 pcs
60031354	HySolate Black Printed x-heavy 152 x 152, 36 pcs
60031353	HySolate Black Printed heavy 152 x 152, 36 pcs
60031352	HySolate Black Printed medium 152 x 152, 36 pcs
60031351	HySolate Black Printed thin 152 x 152, 36 pcs
60033844	HySolate Black Printed thin 152 x 152, 2 pcs
60033845	HySolate Latex Dental Dam Green medium 152 x 152, 2 pcs

Warnings and Precautions

Warnings	Statement
 WARNING!	Failure to properly follow the directions in this IFU may result in patient injury or compromise the integrity of the dental dam. Use the product only in accordance with its intended use, see Intended Use.
	Do not reuse Dental Dam. Dental Dam is intended to be used only once. Reuse of Dental Dam intraorally is a biological hazard and can result in cross contamination.
	Dental Dam should be inspected prior to use. Discard torn or brittle Dental Dam. Use of Dental Dam that has lost its performance characteristics can result in loss of functionality and compromise the integrity of the procedure.

Precautions	Statement
 CAUTION	This Product Contains Natural Rubber Latex Which May Cause Allergic Reactions.
	Do not expose Dental Dam to high temperatures. Store Dental Dam away from heat sources.
	Dental Dam has a specified shelf life. Do not use the device after the Use-by Date because the device functionality and performance characteristics may be compromised.
	Caution: Federal (USA) law restricts this device to sale by or on the order of a licensed healthcare practitioner
	Do not store near non-latex products.

Additional Information

Note: Dispose of device in accordance with local, regional, national, and/or international regulations. Devices used intraorally should be treated as a biohazard and disposed of properly per local regulations.

Note: Upon request, a hardcopy of the IFU will be supplied free of charge.

For users located in the EU: Any serious incident that has occurred in relation to the use of the device should be reported to Coltène/Whaledent, Inc. and the competent authority of the member state in which the user and/or patient is established.

Symbol Glossary

Symbol	Title and description of symbol
	Caution
 www.coltene.com	Consult electronic instructions for use - Indicates the need for the user to consult the instructions for use
	Medical Device – Indicates the item is a medical device
	Sale Restricted: The purchase of the device is restricted to licensed healthcare practitioners only.
	Use-by Date - Indicates the date after which the medical device is not to be used
	Catalog Number - Indicates the manufacturer's catalogue number so that the medical device can be identified
	Do not re-use - Indicates a medical device that is intended for use on a single patient during a single procedure

Symbol	Title and description of symbol
	CE mark
	Importer - Indicates the entity importing the medical device into the locale
	Manufacturer - Indicates the medical device manufacturer
	Batch code - Indicates the manufacturer's batch code so that the batch or lot can be identified
	Authorized representative in the European Community - Indicates the authorized representative in the European Community
	Contains natural rubber latex - Indicates the presence of dry natural rubber latex as a material of construction within the medical device.

Storage

Store Dental Dam between 32° F – 80° F (0°C – 26°C).

Keep dry and keep away from heat.

Symbol	Title and description of symbol
	Keep dry - Indicates a medical device that needs to be protected from moisture
	Keep away from heat - Indicates a medical device that needs protection from heat
	Temperature Limit - Indicates the temperature limits to which the medical device can be safely exposed. The upper and lower limits of temperature shall be indicated adjacent to the upper and lower horizontal lines.

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Dent4You AG
Coltene/Whaledent Inc.
Coltene/Whaledent Ltd.
Coltene/Whaledent GmbH + Co. KG