



# perfect margin rounded



## Prosthetic finishing with chamfered shape



**PM1**

### Preparation, rounded edge, diamond-coated tip 76 $\mu$ m

First instrument of the ultrasonic sequence, following the rotary phase. Intraculcular dentin preparation and positioning of finishing line.



**PM2**

### Finishing, rounded edge, diamond-coated tip 46 $\mu$ m

Correction of irregularities in the finish line and start of polishing. Its diamond coating, less dense than on the PM1, makes it possible to obtain a cutting edge finish.



**PM3**

### Polishing, rounded edge, smooth

This entirely smooth instrument is last in the finishing sequence, improving the condition of the surface at the cervical limit before impression taking.



**PM4**

### Corono-radicular preparation, conical, diamond-coated 46 $\mu$ m

After the rotating phase, the PM4 tip is used to:

- Prepare the upper 1/3 of canal chamber.
- Shape anatomically the connection cone.
- Clean the root walls.
- Smooth the entry cones for the anatomical posts.